
# CANE BUDDY

The Portable, Lightweight Cane with Built-in Lights!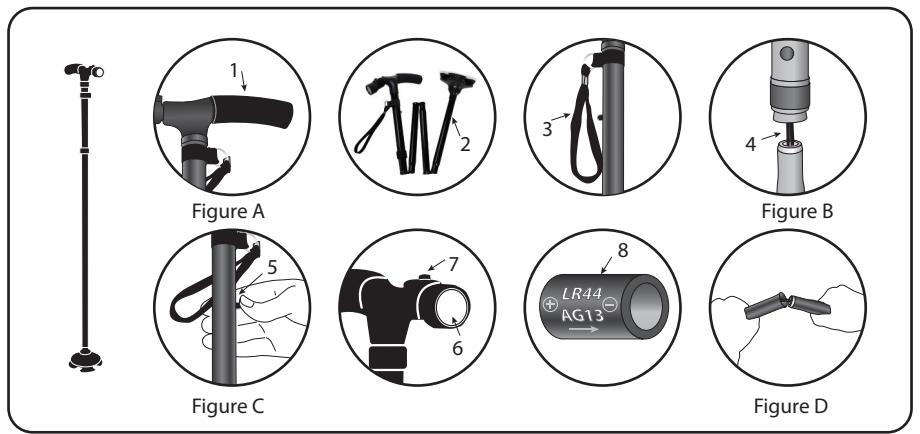

## English

### Product Description

1. Handle
2. Cane Sections
3. Wrist Strap
4. Elastic cord
5. Length adjustment button
6. Light
7. Light Switch
8. Battery compartment

### Instructions for use

#### Unfolding Cane Buddy

Cane Buddy™ has a wrist strap that keeps the sections of the cane together when the cane is folded up. To unfold the cane, first unfasten the wrist strap (3). Grasp the handle (1), hold it up and let the sections of the cane (2) unfold by the force of gravity.

**Important:** To prevent falls, always ensure that all of the cane sections (2) are perfectly aligned and locked in place (see Figure B). Before using the cane for support, make sure that the sections are perfectly aligned and the cane is stiff.

**CAUTION** Uncontrolled release of the elastic cord (4) inside the cane can cause severe injury to unprotected body parts, particularly the eyes. If the elastic cord (4) breaks, becomes damaged or wears out, stop using the product. Do not stretch the elastic cord (4) more than is necessary to unfold or fold up the cane (2).

#### Adjusting the Length of Cane Buddy™

Cane Buddy™ has five different length settings. To make it longer, hold down the length adjustment button (5) as shown in Figure C, pull up on the handle and release the adjustment button to let it lock into a lower slot. To make the cane shorter, hold down the length adjustment button (5), push down on the handle and release the adjustment button to let it lock into a higher slot. Carefully adjust the cane's length to suit your needs.

**CAUTION** To prevent falls, always ensure that all of the cane sections (2) and the length adjustment button are perfectly aligned and locked into place.

#### Using the Light on Cane Buddy™

An adjustable light (6) is located at the end of the handle (1). To turn the light on and off, press the light switch (7) on the cane handle (1). The light pivots up and down so you can point it in the direction you wish.

#### Folding up Cane Buddy™

To store the cane, separate the sections as shown in Figure D and group them together. Once you have separated and gathered all the sections, fasten them with the wrist strap (3). Secure the sections with the fasteners at the each end of the strap.

#### Inserting and Changing the Batteries

Cane Buddy™ operates with 4pcs LR44 batteries. To insert the batteries, unscrew the light (6), pull the tab to remove the battery compartment (8) and insert the batteries. Make sure that the battery poles correspond to the markings on the compartment. Put the battery compartment back in place (6). Make sure the light is secure and working. If it is not, repeat the process and make sure that all of the parts fit perfectly together.

#### Safety Warnings

- Read the instruction manual before using Cane Buddy™ for the first time.
- Always consult your doctor before using Cane Buddy™
- The product is not intended for use in icy or slippery conditions.
- The product is not intended for use by persons weighing over 250 lbs / 115kg.
- Do not disassemble the body of the cane, which contains the elastic cord.
- The product is not for medical use.
- Not suitable for skiing.
- Use areas: asphalt and forest paths. Never use the cane on wet, slippery surfaces.
- This product does not aim to be a walking aid as per EN 1985 (on hiking sticks.)
- Do not use the cane if it is defective.

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